


# **Home & Office Centration Chart**

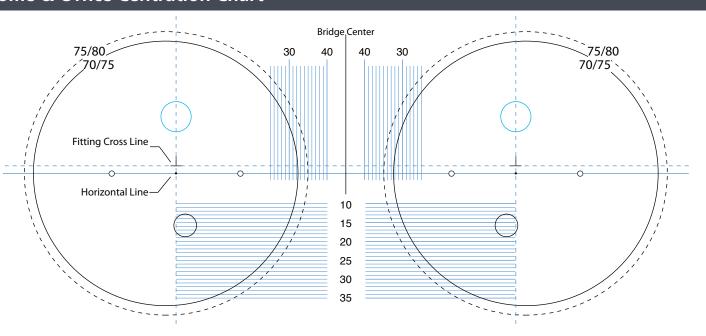

#### **Recommended Fitting Height**

# Minimum Fitting Height: 16 mm 9mm 25mm 13mm 3mm\_

#### **Ordering Nikon Home & Office**

Nikon Home & Office can be ordered like a progressive lens. Each lens is based on the patient's own reading power, (no degression) so there is nothing to calculate.

#### Simply provide the following:

- Far monocular PD
- Add power
- Distance prescription
- Fitting height

### Why a Patient Should be Recommended Nikon Home & Office

- Requires comfortable and wide intermediate vision Experiences visual discomfort in mid-range distance
- Uses various electronic devices at different distances
- Needs an ideal complement to progressive lenses

#### **Ideal Lens for the Following Patients**

Office workers, people working from home, physicians, surgeons, dentists, chefs, hairdressers, musicians, customer service agents, architects, accountants, seamstresses, artists, assembly line workers, electricians, air traffic controllers, technicians, etc., in other words all who use intermediate vision indoors.

Even if a small portion of the lens is devoted to far vision, Nikon Home & Office should not be worn for any far vision activity (e.g. driving).

## www.nikonlenswear.us



Nikon is a registered trademark of Nikon Corporation Transitions and the swirl design are registered trademarks of Transitions Optical, Inc. © 2016 Nikon Optical USA, Inc